



SARS-CoV-2 Antigen Rapid Test

Product Features


- High Accuracy, Specificity and Sensitivity
- Test results within 15-30 minutes
- 12 Months Shelf Life
- Room Temperature Storage
- Oropharyngeal Swab and Nasopharyngeal Swab Available
- Control Set Available

Clinical Performance


Relative Sensitivity: 92.30%

Relative Specificity: 99.40%

Accuracy: 98.98%

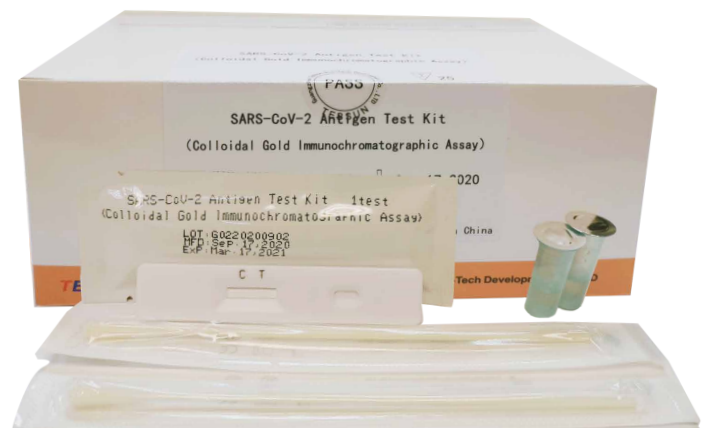

Sampling

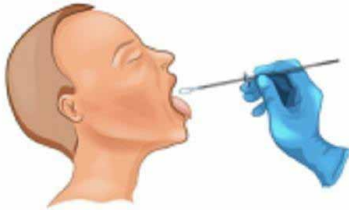

Oropharyngeal Swab




Nasopharyngeal Swab

Test Procedure

1. Rip the aluminum foil pouch to take out the cassette, and put it on a clean place.
2. Specimen collection: according to standard operating methods.
3. Extraction specimen: insert the swab into an extraction reagent tube. Squeeze and stir the swab for more than 5 times.
4. Squeezing the sides of the tube when remove the swab to extract the liquid from the swab.
5. Press the nozzle cap tightly into the tube, and add 2 drops of extracted specimen to the specimen well of the cassette.
6. Observe the results in 15 minutes. The test is invalid clinically after 30 minutes.

Interpretation of Test Results

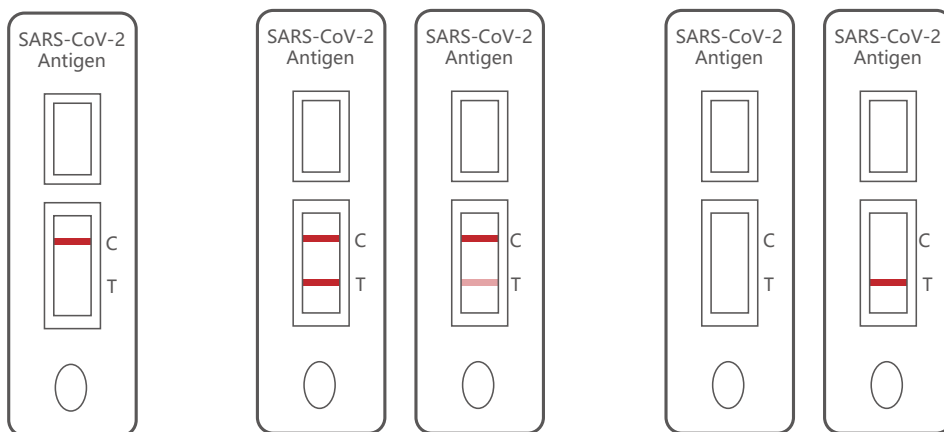

Negative

Positive

Invalid

Guangzhou Tebsun Bio-tech Development Co., Ltd

Address Room 2415,24/F,CITIC Plaza, 233 Tianhe North Road,Tianhe District,Guangzhou

Tell:400-888-4840

Websit:www.tebsun.com



TEBSUN
天宝颂原

